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Sáragel til meðhöndlunar á nýlegum og eldri örum

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medical

SCAR HEALING GEL

A topical gel for the treatment of new and old scars

- Minimizes scar formation
- Reduces existing scars
- Forms a film to protect skin
- Promotes natural skin healing
- Protects against infections
- Reduces redness, itching and pain

Please read the instructions carefully before using this product as it contains important information. Please contact your healthcare professional if you have any questions.

ChitoCare® medical contains natural chitosan, a safe bioactive polymer harvested from the pristine waters around Iceland. Chitosan consists of amino sugars which occur naturally in the human body, and therefore has excellent biocompatibility.

WHY CHITOSAN?

Chitosan is a functional polysaccharide providing support and under mild acidic conditions it becomes positively charged, rendering it bioadhesive to adhere as a protective film to intact or breached skin and to confer many other important biological properties. Chitosan is a true wonder from the sea as it protects the skin, reduces redness and irritation with its cooling effect, maintains moisture and promotes the skin's natural repair process.

INDICATION AND INTENDED USE

For the treatment of new and old scars. The Scar Healing Gel provides a soothing effect and support to the scar tissue to heal in an orderly manner. It protects against infections, stimulates cell proliferation in old and new

scars, as well as correct tissue organization with proper collagen alignment for a better aesthetic appearance and skin flexibility. Ideal for treating scars after accidents, surgery, C-section and cosmetic surgery. For smoother and less visible scars. May be used by the whole population, also elderly and children.

INSTRUCTIONS FOR USE

After stitches removal, apply a thin layer to the scar area and let dry for 2 min. A transparent and elastic film will form to protect your skin. Use twice a day or as needed. Note that the chitosan film does not need to be removed as it will slowly be degraded during natural skin renewal. Before using for the first time, pump the airless bottle a few times until a drop appears. The bottle is then ready for use. Wash your hands before applying the gel to the scar area. Replace the cap after use to protect the dispenser.

UNDESIRABLE SIDE-EFFECTS

No side-effects are known unless allergy to an ingredient causes skin irritation or other reactions. Always apply first to a small skin area to test product suitability.

PREGNANCY AND BREASTFEEDING

There are no data available on the use of this product during pregnancy or breastfeeding. However, it contains natural ingredients, known to the body metabolism, that will be naturally degraded on the skin.

USE FOR CHILDREN

Can be used safely on children..

INGREDIENTS

Deionized water, ChitoClear® chitosan, lactic acid, sodium benzoate
Contains chitosan from shrimp shell. Allergy studies show no skin reactions.

 Precautions: For external use only. Keep out of sight and reach of children. Do not use if you are allergic to one of the ingredients. Always wash hands before product application. Make sure the dispenser head does not come in contact with breached or infected skin. Avoid contact with eyes. In case of accidental contact, rinse with an abundant amount of water. This product is for multiple use; the primary packaging is designed to prevent contamination. After the expiration date, do not use the product and dispose of it as normal waste.

 See expiry date under the packaging (YYYY-MM).

STORAGE CONDITIONS

 Keep between 2-25°C, away from sunlight.

 Best stored refrigerated.

 Use within 8 months of opening.

 Do not use the product if the packaging is damaged.

ChitoCare® medical Scar Healing Gel 30 ml, Ref.no. 84100. This product is a medical device in accordance with the European Directive 93/42/EEC on Medical Devices.

Last revised in April 2021 (8374L01)

primex
ICELAND

 PRIMEX EHF.

ÓSKARSGATA 7 · 580 SIGLUFJORDUR · ICELAND
TEL: (+354) 460 6900 · MEDICAL@CHITOCARE.IS · CHITOCARE.IS