SÁRAGEL

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ChitoCare® medical Sáragel 30 ml, Ref. no. 84300.

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medical

WOUND HEALING GEL

A topical gel for acute and chronic wounds and other skin conditions

- Effectively reduces chronic wounds
- Protects against infections
- Heals wounds and burns
- Reduces redness, itching and pain
- Helps reduce bleeding
- Minimizes scarring
- Promotes natural skin healing

Please read the instructions carefully before using this product as it contains important information. Please contact your healthcare professional if you have any questions.

ChitoCare® medical contains natural chitosan, a safe bioactive polymer harvested from the pristine waters around Iceland. Chitosan consists of amino sugars which occur naturally in the human body, and therefore has excellent biocompatibility.

WHY CHITOSAN?

Chitosan is a functional polysaccharide providing support and under mild acidic conditions it becomes positively charged, rendering it bioadhesive to adhere as a protective film to intact or breached skin and to confer many other important biological properties. Chitosan is a true wonder from the sea as it protects the skin, reduces redness and irritation with its cooling effect, maintains moisture and promotes the skin's natural repair process.

INDICATION AND INTENDED USE

For the treatment of acute and chronic wounds, and other skin conditions. The Wound Healing Gel protects the skin and wounds against infections, helps reduce bleeding, scarring and pain due to its cooling effect. It soothes the symptoms associated with various skin conditions, such as itching and redness, promoting the natural skin repair process. Ideal for sunburns,

scars, pimples, acne symptoms, cold sores, blisters, skin rash, eczema, dry and cracked skin, bug bites and skin inflammation. It can also effectively reduce acute and chronic wounds. Can be used on cuts, abrasions, burns, pressure sores, diabetic ulcers, and other chronic wounds. Essential first aid. May be used as a preventive treatment to protect the skin. May be used by the whole population, also elderly and children.

INSTRUCTIONS FOR USE

Apply a thin layer to the affected area and let dry for 2 min. A transparent and elastic film will form to protect the skin and wounds. Use twice a day as needed or recommended by your doctor. May be covered with a secondary dressing if necessary. Note that the chitosan film does not need to be removed as it will slowly be degraded during natural skin renewal. Before using for the first time, pump the airless bottle a few times until a drop appears. The bottle is then ready for use. Wash your hands before applying the gel to the wound. Replace the cap after use to protect the dispenser.

UNDESIRABLE SIDE-EFFECTS

No side-effects are known unless allergy to an ingredient causes skin irritation or other reactions. Always apply first to a small skin area to test product suitability.

PREGNANCY AND BREASTFEEDING

There are no data available on the use of this product during pregnancy or breastfeeding. However, it contains natural ingredients, known to the body metabolism, that will be naturally degraded on the skin.

USE FOR CHILDREN

Can be used safely on children. Recommended for diaper rash and eczema.

INGREDIENTS

Deionized water, ChitoClear® chitosan, lactic acid, sodium benzoate
Contains chitosan from shrimp shell. Allergy studies show no skin reactions.

 Precautions: For external use only. Keep out of sight and reach of children. Do not use if you are allergic to one of the ingredients. Always wash hands before product application. Make sure the dispenser head does not come in contact with breached or infected skin. Avoid contact with eyes. In case of accidental contact, rinse with an abundant amount of water. This product is for multiple use; the primary packaging is designed to prevent contamination. After the expiration date, do not use the product and dispose of it as normal waste.

 See expiry date under the packaging (YYYY-MM).

STORAGE CONDITIONS

 Keep between 2-25°C, away from sunlight.

 Best stored refrigerated.

 Use within 8 months of opening.

 Do not use the product if the packaging is damaged.

ChitoCare® medical Wound Healing Gel 30 ml, Ref. no. 84300. This product is a medical device in accordance with the European Directive 93/42/EEC on Medical Devices.

Last revised in April 2021 (8275L01)

 **primex**
ICELAND

 PRIMEX EHF.

ÓSKARSGATA 7 · 580 SIGLUFJORDUR · ICELAND
TEL: (+354) 460 6900 · INFO@PRIMEX.IS · CHITOCARE.IS