ChitoCare

medical

GRÆÐANDI SPREY

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ChitoCare® medical Græðandi Sprey 50 ml, Ref. no. 84500.

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medical

HEALING SPRAY

A topical spray for irritated and damaged skin

- Protects skin
- Heals minor wounds and burns
- Reduces redness, itching and pain
- Relieves minor burns and sunburns
- Promotes natural skin healing

ChitoCare® *medical* contains natural chitosan, a safe bioactive polymer harvested from the pristine waters around Iceland. Chitosan consists of amino sugars which occur naturally in the human body, and therefore has excellent biocompatibility.

Please read the instructions carefully before using this product as it contains important information. Please contact your healthcare professional if you have any questions.

WHY CHITOSAN?

Chitosan is a functional polysaccharide providing support and under mild acidic conditions it becomes positively charged, rendering it bioadhesive to adhere as a protective film to intact or breached skin and to confer many other important biological properties. Chitosan is a true wonder from the sea as it protects the skin, reduces redness and irritation with its cooling effect, maintains moisture and promotes the skin's natural repair process.

INDICATION AND INTENDED USE

For the treatment of irritated and damaged skin. The **Healing Spray** soothes the symptoms associated with these conditions, such as itching and redness, protects the skin and promotes its natural repair process. Ideal for minor wounds, minor burns, sunburns, pimples, acne symptoms, skin rash, dry skin, bug bites and skin inflammation. **Essential first aid.** May be used on intact or breached skin, and as a preventive treatment to protect the skin. May be used by the whole population, also elderly and children.

INSTRUCTIONS FOR USE

Apply a thin layer to the affected area by spraying evenly from about 5–10 cm distance and let dry for 2 min. A transparent and elastic film will form to protect the skin. Use twice a day or as needed. Alternatively, spray onto a clean hand or finger and apply where needed.

Note that the chitosan film does not need to be removed as it will slowly be degraded during natural skin renewal.

Before using for the first time, spray a few times until a uniform mist. The bottle is then ready for use. Wash your hands before applying the spray to the affected area. Replace the cap after use to protect the dispenser.

UNDESIRABLE SIDE-EFFECTS

No side-effects are known unless allergy to an ingredient causes skin irritation or other reactions. Always apply first to a small skin area to test product suitability.

PREGNANCY AND BREASTFEEDING

There are no data available on the use of this product during pregnancy or breastfeeding. However, it contains natural ingredients, known to the body metabolism, that will be naturally degraded on the skin.

USE FOR CHILDREN

Can be used safely on children. Recommended for diaper rash.

INGREDIENTS

Deionized water, ChitoClear® chitosan, lactic acid, sodium benzoate
Contains chitosan from shrimp shell. Allergy studies show no skin reactions.

Precautions: For external use only.

Keep out of sight and reach of children.

Do not use if you are allergic to one of the ingredients.

Always wash hands before product application.

Make sure the dispenser head does not come in contact with breached or infected skin.

Avoid contact with eyes. In case of accidental contact, rinse with an abundant amount of water.

This product is for multiple use.

After the expiration date, do not use the product and dispose of it as normal waste.

See expiry date under the packaging (YYYY-MM).

STORAGE CONDITIONS

Keep between 2–25°C, away from sunlight.

Best stored refrigerated.

Use within 8 months of opening.

Do not use the product if the packaging is damaged.

ChitoCare® *medical* Healing Spray 50 ml, Ref. no. 84500. This product is a medical device in accordance with the European Directive 93/42/EEC on Medical Devices.

Last revised in April 2021 (8305L01).

primex
ICELAND

PRIMEX EHF.

ÓSKARSGATA 7 · 580 SIGLUFJORDUR · ICELAND
TEL: (+354) 460 6900 · INFO@PRIMEX.IS · CHITOCARE.IS